

SURFA'SAFE Premium



Disinfecting detergent solution for surfaces and medical devices

Disinfecting detergent solution

Disinfectant detergent solution for cleaning and disinfecting non-invasive, non-immersible medical devices. Biocide application: for general surfaces.

SURFA'SAFE PREMIUM is a ready-to-use surface cleaning and disinfection product.

Combining broad spectrum efficacy, short contact times with excellent material compatibility (including alcohol sensitive surfaces) and the properties of a gentle cleaner.

Its features make SURFA'SAFE PREMIUM suitable for use on a wide range of surfaces in healthcare facilities and an ideal product for intensive care and other high-risk areas.

This enables the products to be used in practice as intended in your protocols, delivering the disinfection results and compliance level mandatory for effective disinfection to restrict the spread of pathogens.

- Good foaming properties.
- Efficient formula without unnecessary ingredients (perfume and dye free).
- Cleaning and disinfection in one step, leaving the surface visibly clean.



SURFA'SAFE Premium

Disinfectant detergent solution, intended for cleaning and disinfecting non-invasive and non-immersible medical devices.

Biocide application: for general surfaces.



SURFA'SAFE Premium
750ml Spray Bottle

Product characteristics

- Cleans and disinfects in one step
- Without alcohol, perfume and dye
- Broad spectrum antimicrobial and optimised contact time
- Formula ensures compatibility with a wide range of materials, including polymers (e.g. polycarbonate, plexiglas)
- Compact foam dispenser to limit the formation of aerosols and volatile components (750 ml)
- Dense foam for better surface adhesion (750 ml)
- pH 6

Microbiological properties

Claim/Conditions	Contact time
Bactericidal - According to: EN 13727* (1 min), EN 16615* (2 min)	2 min
Yeasticidal - According to: EN 13624* (2 min), EN 16615* (2 min)	2 min
Tuberculocidal - According to EN 14348*	60 min
Active against enveloped viruses according to EN 14476*	1 min
Active in accordance with EN 14476* against Polyomavirus SV 40	5 min
Active in accordance with EN 14476* against Rotavirus	30 sec
Limited spectrum virucidal according to EN 14476*	30 min
EN 14476* Adenovirus	30 min
EN 14476* Norovirus (MNV)	15 min

* dirty conditions

Packaging and ordering information

Product	Packaging unit	Order code
SURFA'SAFE Premium	12 x 750 ml	XXXXX

Application area

Disinfectant detergent solution for cleaning and disinfecting non-invasive and non-immersible medical devices.

Biocide application: for general surfaces.

Instructions for use

Ready-to-use solution.

For sprayer:

Spray the disinfectant detergent solution on a non-woven cloth and wipe.

For other packaging:

Pour the disinfectant detergent solution on a non-woven cloth and wipe.

Applicable for all pack sizes:

Spread evenly the product (+/- 6 ml/m² or around 10 sprays).

Respect the indicated contact time for the required antimicrobial activity.

Rinsing is unnecessary, except if treatment areas are intended to come into contact with skin, mucosa or foodstuffs.

Please refer to the manufacturer's instructions for use for cleaning and disinfecting medical devices.

For the frequency of use and cleaning of the application equipment, refer to the hygiene plan in place. Scientific dossier (efficacy, compatibility, etc.) available on request.

Ingredients

0,3 g of Didecyltrimethylammonium chloride (DDAC) (CAS No. 7173-51-5), non-ionic surfactants, chelating agent, excipients.

Storage

Storage: between +5°C and +25°C.

Medical device and Biocides use safely

Please read label and product information.



FOR PROFESSIONAL USE ONLY.